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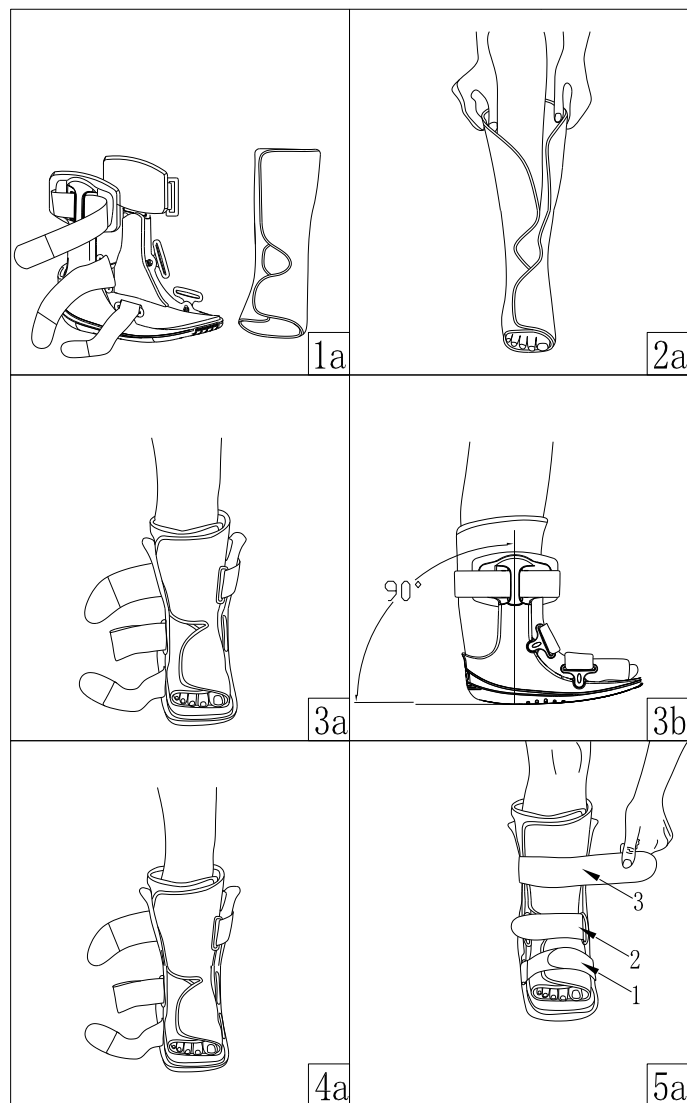
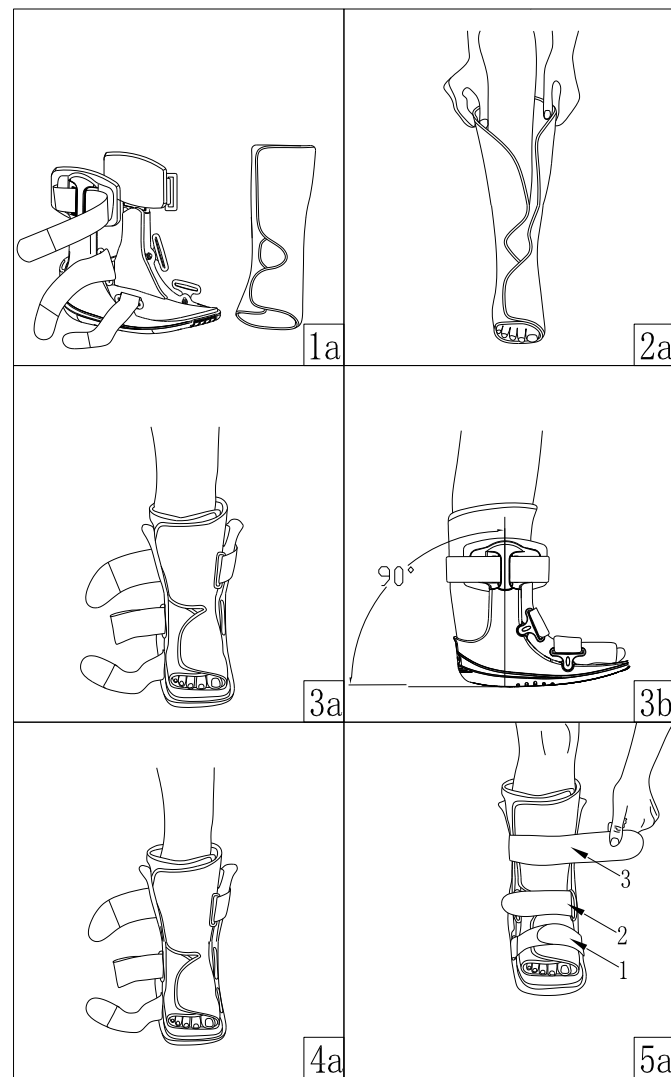
### 4)Product Structure:

The product is mainly composed of base, Strut, straps, lining etc

### 5)Operating principle:

The main body of the product is provided with support, anti-slip soft sole and shock absorption system. The ankle and calf are properly protected through the support. Through software and shock absorption system, patients have more comfort in their legs during exercise. The shoe body big body and the calf combination, causes the patient leg to obtain the perfect protection. The breathable soft shoe pad enables the product to provide comprehensive protection for the feet before and after use, to prevent secondary injury, and to provide soft contact between the calf and the fixed support to make walking more comfortable. Can give the patient's foot a very good buffer, help the patient get a good recovery.

### Scope of application:

External use, provides support for foot fracture, foot ligament injury fixation, foot sole and toe injury fixation, mild and moderate foot thumb valgus, plantar pain, postoperative rehabilitation

### Product features and performance description

1) Product features and storage conditions

**【Product features】** Low profile rocker sole, Shock-absorbing technology, High strength & light weight, Mechanical strap locking system.

**【Storage conditions】** This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2)Using the operating **【Application Instructions】**

**Note: Do not contact the liner directly with the skin.**

**\*Please purchase and use the product under the guidance of your doctor or professional.**

**STEP 1:** Release the nylon buckle and remove the internal gasket (Fig1a).

**STEP 2:** place the foot in the pad, heel close to the back of the pad, wrap the foot and leg. Make sure the legs and feet are close to the gasket (Fig 2a).

**STEP 3:** gently expand the side pillar and place the calf in the bracket while the pull strip is intact. The ankle should be concentrated on the pillar, with the foot and ankle at 90 ° ( Fig3a~3b).

**STEP 4:** a After correct alignment, adjust the straps and gaskets to know the comfortable position (Fig 4a)

**STEP 5:** Fasten the strap 1-2-3 from the bottom to the top of the toe, pay attention to the hole position of the strap and the column position on the support frame can not be separated, fasten the strap not too tight. . As the swelling pattern changes over the day, it may be necessary to adjust the tightness of the strap for comfort(Fig 5)

**STEP 6:** To remove, simply release the tape and pad and pull the

heel and ankle forward to reapply the foot and ankle into the gasket and reconnect the buckle (Fig 5a)

### IMPORTANT:

- 1) Do not contact the liner directly with the skin.
- 2) Warnings and Over tightening or improper bandaging may cause severe skin irritation, especially for diabetic neuropathy (a diabetic neurological condition). Always check for "hot spot" and skin irritation on your skin. If you find any skin changes or feel uncomfortable, loosen the tightness of the dressing. Do not use the device if it is not felt due to post-operative anesthesia, nerve, skin, or other conditions.

### Maintenance:

The software part can be washed with a neutral detergent and a 30 & deg; C sailor and air-dried. Do not put the dryer or use other heat sources for drying. Elements such as salt water, sand, dirt, etc. may cause injury. After exposure to these conditions, rinse thoroughly with clear water, dry the towel or air-dry (do not machine the machine). For detergents, use mild detergent and do not use softeners and bleaches.

The rigid part can be scrubbed with wet cloth and dried naturally.

### Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

\*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

### Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

### Product material

- 1) Base/ strut: polypropylene+GF
- 2) Straps: nylon
- 3)Pad: nylon / polyurethane / polyester (outer net / foam / inner mesh)
- 4)Shoehoutsole/shoe inner sole: thermoplastic elastomer / polypropylene/EVA

### Contraindication

- ⚠ Use with caution for unstable fractures;
- ⚠ Use with caution in osteoporosis;
- ⚠ Use with caution for cognitive impairment;
- ⚠ Use with caution for bony ankylosis;

### Storage, transportation and expiry date of products

- 1)Product is valid: five years
- 2)Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3)Transportation: the product shall be dust-proof, moisture-proof and pollution-proof during storage and transportation

### Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

### Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- CE Symbol
- Hand wash only
- Protect from sunlight
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

### Accessories list

Instructions ----- 1 copy

Pad ----- 3 pcs

### After-sales warranty

Warranty: all products from the date of purchase as the original date, normal wear and tear of the product in the use of the process is not considered defective.

The following parts from the date of purchase to the warranty period, if there is any material or process defects, please contact your seller directly, Simple medical will complete or partial repair or replacement of equipment and accessories.

### Simple Medical (Shenzhen) Limited

Address: 519 Wisdom Valley, No.1010 Bulong Road,

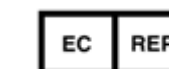
Minzhi Street, Longhua District,

Shenzhen, China.518109

Tel : +86-755-86038563

Web: www.simplemedical.cn

E-mail: info@simplemedicla.cn`

SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

\*Please refer to the Instructions before usage

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference.