
## Simple Medical(Shenzhen)Limited Instruction For Use

### Knee Immobilizer K4

Model: 11022 Size: S/M/L/XL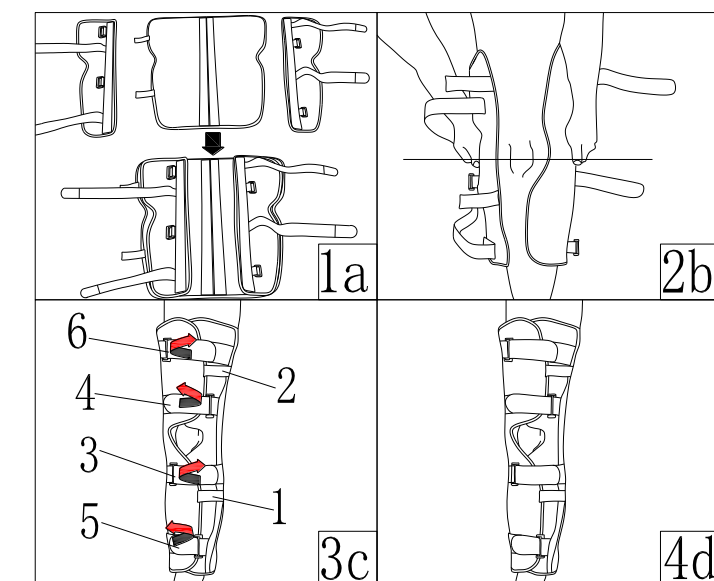

#### Product Structure and Performance

Knee retainer is mainly composed of breathable and soft body, support, strap and tie buckle. The products are mainly used for emergency knee fixation, knee fixation before and after operation and traumatic knee injury. The product adopts streamline lightweight design. The outer side of the product is provided with a support, through which the knee joint and the large / calf are properly protected and stable. Detachable fixed edges make patients more convenient and flexible to use.

#### INDICATIONS

- Fixation of knee ligament strain
- Meniscus injury fixation
- Fixation of patella fracture
- Postoperative rehabilitation

#### APPLICATION INSTRUCTION

##### STEP 1.

Loosen the fastening After loosening the fixing belt, the fixed edges on both sides can be removed and re-fixed according to the needs of the individual (please fasten both sides firmly) (Fig 1a).

##### STEP 2.

The center of the gap on both sides is aligned with and wrapped around the center of the knee (Fig. 2b)

##### STEP 3.

Fasten the straps. Hold both sides with your hands to prevent movement. And fasten the straps 1/2 in turn. Then tie the strap 3 / 4 / 5 / 6 (around the leg through the strap buckle and fasten it in the opposite direction (Fig3c-4d)

#### NOTICE:

Fasten the strap on upper thigh, not too tight. Too tight may lead to slow blood circulation or downward movement of the bracket. Adjust the position of the straps and bracing if necessary.

#### "Wearing time" :

Adjust the wearing time of the brace when the body allows it.

#### Preoperative wearing time:

The wearing standard and wearing time of the knee retainer should be set by the doctor until your attending doctor agrees to remove the brace.

**\*Please purchase and use the product under the guidance of your doctor or professional.**

#### \* IMPORTANT :

- 1) The product is for use by a single patient and is not recommended for multiple patients.
- 2) The knee joint and this product should be checked before and after each use. This product should be used in good skin.
- 3) This device cannot prevent or reduce all injuries, and proper rehabilitation and activity improvement are also an indispensable process in the safety treatment plan.

#### CARE AND MAINTENANCE

Liners can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying

Metal parts should not be washed with water.

#### WARNINGS:

If you feel pain, swelling, change of sensation, or discomfort when using this product, please stop using it and contact your doctor.

#### Basic application environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2)Relative humidity:≤ 80%

#### Product materials

- 1) Support: aluminum alloy
- 2) Buckle: nylon
- 3) Accessories: Nylon
- 4) Straps/ main body: nylon/ polyurethane/ polyester/ EVA

#### Contraindication

- ⚠ Use with caution for unstable fractures;
- ⚠ Use with caution in osteoporosis;
- ⚠ Use with caution for cognitive impairment;
- ⚠ Use with caution for bony ankylosis;

#### Storage, transportation and expiry date of products

- 1) Term of validity: Three years.
- 2) Storage of the product: please put it in a dry and clean room before use to ensure indoor ventilation.
- 3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation.

#### Waste disposal

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

#### Symbol and picture description

	Manufacture Date
	Batch Code
	medical device
	CE Symbol
	Protect from sunlight
	Hand wash only
	Do not use when the packing is damaged
	Warnings and Precautions
	Reference instructions

#### Accessories list

Instructions ----- 1 copy

#### After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street, Longhua District, Shenzhen, China.518109  
Tel : +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: info@simplemedicla.cn`

**EC REP** SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

\*Please refer to the Instructions before usage

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