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【网址】www.simplemedical.cn

Simple Medical (Shenzhen) Limited Instruction For Use

Arm Sling Shoulder Brace E5

Model: 31801 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

1)Product name: Arm Sling Shoulder Brace E5

2)Product model: 31801

3)Product specification: F

4) Product Structure:

The product is mainly composed of main body pocket, shoulder strap, 30° shoulder extension pillow, belt, grip ball, buckle, double-sided magic strip and magic paste.

5) Operating principle

The product is open-ended designed to prevent internal rotation by fixing the patient's arm in a neutral position. The shoulder belt, the waistband and the waist form a triangular structure, and the neck pressure is effectively reduced. The detachable holding force rehabilitation ball is convenient to use and can exercise the gripping force of the hand finger at any time. The strap and the belt are in a snap-fit connection to make the patient wear more convenient.

6) Scope of application:

It is suitable for trigonometry, upper tendon rupture, shoulder and arm joint fracture and dislocation, shoulder and arm joint operation support, brachial plexus injury.

Product features and performance description

1)Product features and storage conditions

【Product features】


Fixation with shoulder abduction, Easy fit and convenient adjust

【Storage conditions】

This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating

【Application Instructions】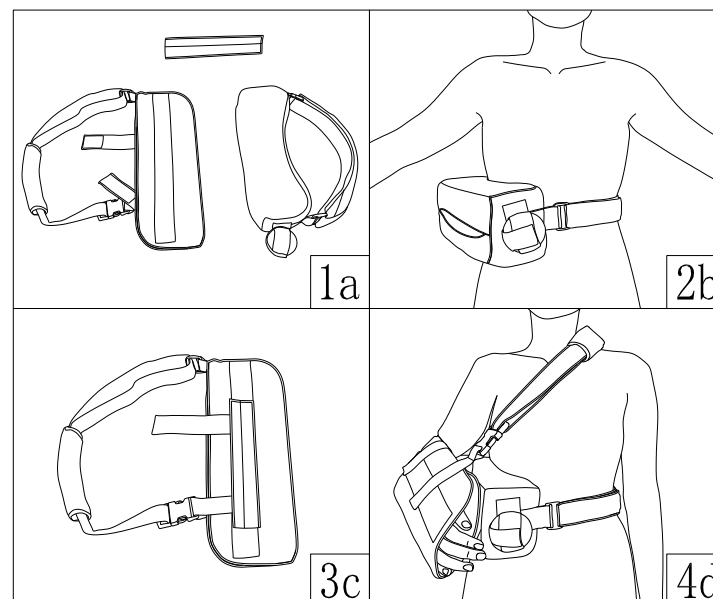

APPLICATION INSTRUCTIONS

STEP 1: Open the product (Fig. 1a)

STEP 2: Loosen the buckle of the belt, place the shoulder pillow at the waist, and connect the buckle. Adjust the length of the belt so that the belt fits with the waist (Fig. 2b).

STEP 3: Stick the double-sided magic sticker on the hair surface of the main pocket near one side of the body (Fig. 3 c).

STEP 4: Loosen the two straps and shoulder straps on the outside of the sling pocket, place the hand in the sling pocket, fix the fastening belt on the hairy surface of the sling pocket, place the sling on the shoulder spread pillow, connect the shoulder strap buckle through the shoulder, and adjust the shoulder belt to the appropriate length so that the shoulder pad is located at the shoulder (Fig4 d).

Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach. The ccipital core in the external pillow of the shoulder cannot be cleaned

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors

Basic product environment

1) Ambient temperature: -15 ~ 60 °C

2) Relative humidity: 80%

Product material

1) Shoulder strap: nylon/ sponge

2) Main body suspension: nylon/ sponge

3) shoulder abduction pillow: nylon/ sponge

4) Buckle: Polyethylene

5) grip ball: EVA

6) Double-sided magic patch: nylon/ aluminum alloy

Contraindication.

⚠ Do not use in case of allergy;

⚠ Those with unknown skin masses are forbidden;

⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

1)Product is valid: Three years

2)Product storage: Please place it in a dry and clean room before use to ensure ventilation

3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation.

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

Manufacture Date

Batch Code

medical device

CE Symbol

Hand wash only

Protect from sunlight

Do not use when the packing is damaged

Warnings and Precautions

Reference instructions

Accessories list

Instructions ----- 1 copy

Outer shoulder pillow 30°----- 1 PCS

Grip ball----- 1 PCS

Double sided magic sticker----- 1 PCS

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street, Longhua District,
Shenzhen, China.518109
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: info@simplemedicla.cn`

EC REP
SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

***Please refer to the Instructions before usage**