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【网址】www.simplemedical.cn

# Simple Medical (Shenzhen) Limited Instruction for Use

## Wrist brace W2

Model:42003/42004 Size: S/M/L/XL

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

### Product overview

- 1)Product name: Wrist brace W2
- 2)product model: 42003/42004
- 3)Product specification: S/M/L/XL
- 4)Product Structure:

The wrist fixed support mainly by breathable soft material, aluminum alloy, plastic parts, fixed belt

#### 5)Operating principle

The product is designed to be lightweight and has two support devices on the surface. The support device and breathable soft fabric are used to improve the wearing comfort of patients. The support plate fitting the wrist contour makes the wrist and the protector have proper support and protection to help relieve the wrist muscle and joint pain, provide support and protection for the injured wrist, avoid re-injury, and effectively ensure the patient to get a good recovery.

#### 6)Scope of application

It is used in patients with Carpal Tunnel Syndrome, wrist joint and wrist muscle pain

### Product features and performance description


#### 1) Product features and storage conditions

【Product features】 Reinforced support. Elastic adjustable fit. Open design, light weight, easy use. Breathable, comfortable

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

#### 2)Using the operating

##### 【Application Instructions】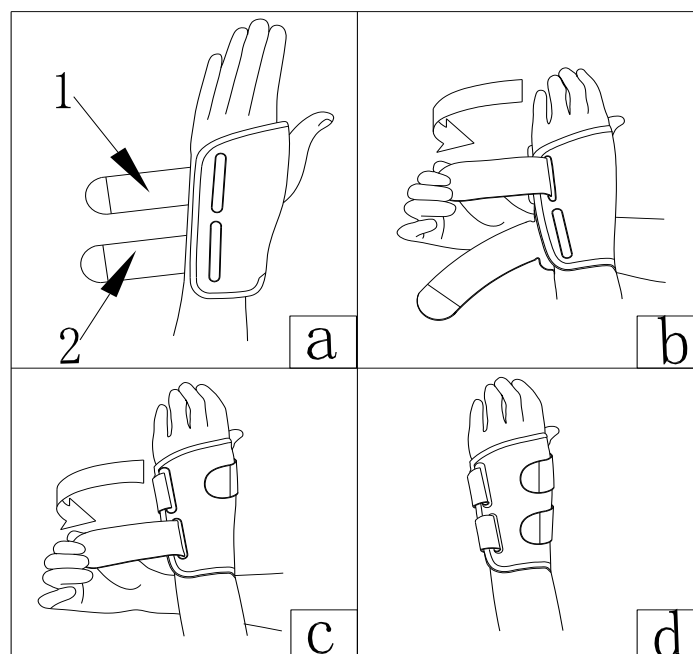

### APPLICATION INSTRUCTIONS

#### STEP 1.

Loosen the two straps of the wrist guard, put your hand into the guard and pass your thumb through the thumb hole (fig a)

#### STEP 2.

Fasten strap# 1(fig b)

#### STEP 3.

Fasten strap# 2(fig c)

#### STEP4.

Strap can be adjusted according to comfort level and supporting effect (fig d)

### Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

### Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

### Product basic parameter

- 1) Ambient temperature: -15 ~ 60°C
- 2) Relative humidity: 80%

### Product materials

- 1) Body material: spandex, nylon, polyester PU
- 2) Support sheet: aluminum alloy/PP
- 3) Buckle: polypropylene
- 4) Strap: nylon/polyurethane/polyester

### Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

### Storage, transportation and expiry date of products

- 1)Product is valid: Three years
- 2 )Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3)Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

### Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

### Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- CE Symbol
- Hand wash only
- Protect from sunlight
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

### Accessories list

Instructions ----- 1 copy

### After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road,  
Minzhi Street, Longhua District,  
Shenzhen, China.518109  
Tel : +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: info@simplemedicla.cn`

**EC REP**  
SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

\*Please refer to the Instructions before usage