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Simple LSO 200

Model: 52009 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

- 1)Product name: Simple LSO 200
- 2)Product model: 52009
- 3)Product specification: F
- 4)Product Structure:

The product is mainly composed of air-permeable mesh cloth, nylon wire, pull ring, liner, plastic pulley support system and magic tape.

5)Operating principle:

The product adopts a double independent mechanical pulley system to provide patients with superior compression level, maximum comfort, flexible control, stable support and effective pain relief. The flexible rear pulley system conforms to the patient's body type and can easily adjust the required comfort level when standing or sitting. A single product has many choices of suitable sizes. The contoured filler panel and breathable mesh provide comfortable wearability for patients.

6)Scope of application:

It is suitable for IDET procedures, Low back pain/strain, Lumbar disc injury, Compression fractures, Osteoporosis, Disc herniation and degeneration, Spondylolisthesis

Product features and performance description


1) Product features and storage conditions

【Product features】 Multi-fit sizing options, double independent pulley system, contoured padded panels, wide adjustments, greater comfort & compliance .

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating

【Application Instructions】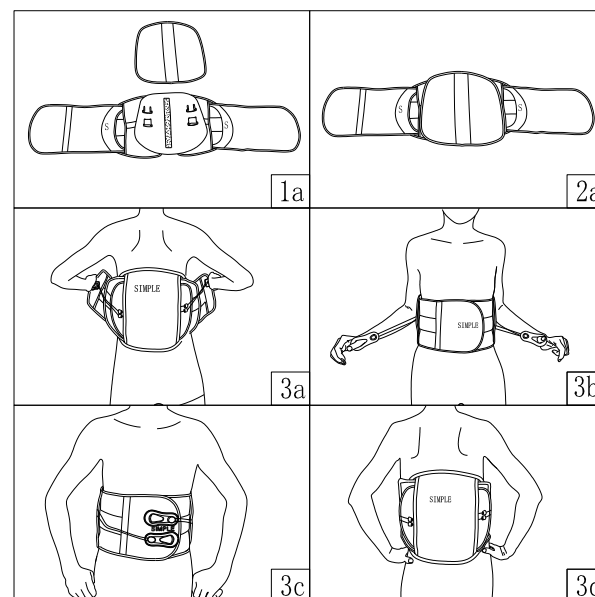

APPLICATION INSTRUCTIONS

STEP 1: Take the soft gasket down, peel off the connecting plate with the transverse extension wide fixed band, and then adjust the wide fixing band through the fixing port and the support port according to the personal body type to the appropriate size (internal marking S/M/L/XL/XXL) fold back into the fixing port with the edge of the middle connection plate as the boundary, and then fix it back into the fixing port. (Fig1a)

STEP 2: Pass the removed gasket through the wide band, and the wool surface of the gasket is aimed at and glued to the hook in the middle of the support (Fig 2a).

STEP 3: Place the Lumbar Brace at the waist, the center is aligned with the spine, and the bottom is at the coccyx. After the fixed belt is fastened to the waist, the hook panel will be affixed to the hairy surface panel. Then pull the pull ring out until the fastening band is tightened and then firmly adhered to the front panel(Fig 3a~3d).

WARNING:

The application instructions listed above are for use only by qualified professionals. Unless otherwise directed by orthopedic professionals, patients are not advised to try to use or adjust the device. Any further adjustment or wear frequency and wear time will be determined by your orthopedic professional.

Maintenance:

Soft items can be washed with neutral detergents and 30 °C sailors and dried naturally. Do not put in the clothes dryer or use other heat sources for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with clean water and dry or dry towels (not machine dry). If you need detergent, please use mild detergent; do not use softener and bleach. Rigid parts can be wiped clean with wet cloth.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

* For a single patient only, not recommended for multiple patients.

Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

Product material

- 1) Main body: nylon.
- 2)pulley support: polyethylene
- 3) pull ring: thermoplastic elastomer
- 4)backing strap :nylon

Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

- 1)Product is valid: Three years
- 2)Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- CE Symbol
- Protect from sunlight
- Hand wash only
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

Accessories list

Instructions----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street,Longhua District,
Shenzhen, China.518109
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: info@simplemedicla.cn`

SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

***Please refer to the Instructions before usage.**