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【网址】www.simplemedical.cn

Simple Medical (Shenzhen) Limited
Instruction for Use

Simple LSO 203

Model: 52004 Size: S/M/L/XL/XXL

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

- 1)Product name: Simple LSO 203
- 2)Product model: 52004
- 3)Product specification: S/M/L/XL
- 4)Product Structure:

The product is mainly composed of air-permeable mesh cloth, nylon wire, pull ring, liner, plastic pulley support system and magic tape.

5)Operating principle:

The product adopts double independent mechanical pulley system, which provides patients with superior compression level, maximum comfort, flexible control system, stable support and effective pain relief. Flexible rear pulley system, in line with the patient's body type, standing or sitting can easily adjust the required comfort. Soft cushions provide comfortable wear for patients.

6)Scope of application:

It is suitable for lumbar discectomy / fusion surgery with low back pain / strain lumbar disc injury, compression fracture, osteoporosis, disc herniation and degenerative lumbar spondylolisthesis.

Product features and performance description


1) Product features and storage conditions

【Product features】double independent pulley system, contoured padded panels, adjustable pull cord handles, comfortable, breathable

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating

【Application Instructions】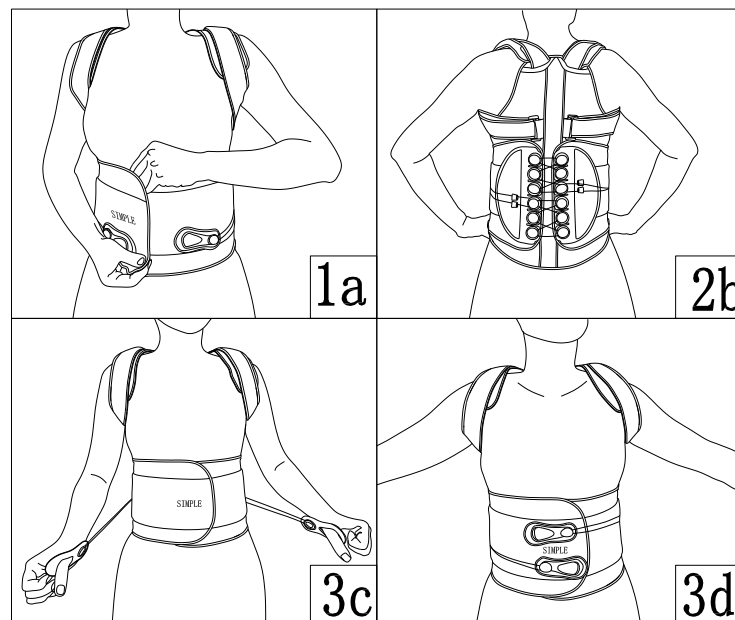

APPLICATION INSTRUCTIONS

STEP 1: Open the two belts in front and put your hands through the straps like a backpack. The center of the dorsal plate is aligned with the spine, and the bottom is about the sacra joint. The hook belt is fastened to the hairy belt so that the waist retainer fits closely with the body (Fig. 1a).

STEP 2: Adjust the shoulder strap. Remove the magic sticker at the end of the shoulder strap and adjust the shoulder strap and shoulder pad to a comfortable position to fit the shoulder strap and pad to the body. If the straps are too long, cut the straps to the desired length, and then install the removed magic stickers and fasten them to the straps (Fig.2b).

STEP 3: After the proper and comfortable wearing position is checked, pull the pull rings on both sides of the wide belt to pull out, so that the fixer is tightened and attached to the body and then adhered to the surface of the wide belt (Fig. 3c~3d).

***CAUTION:**

1. The above steps should be used by qualified professionals. Any further adjustment or wear frequency and wear time will be determined by your orthopedic professional.
2. Don't tighten the pressure system too much to avoid discomfort or dyspnea. When sitting down, the patient may need to loosen the pressure system a little.

Maintenance:

Soft items can be washed with neutral detergents and 30 °C sailors and dried naturally. Do not put in the clothes dryer or use other heat sources for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with clean water and dry or dry towels (not machine dry). If you need detergent, please use mild detergent; do not use softener and bleach. Rigid parts can be wiped clean with wet cloth.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, please discontinue use and consult your doctor.
* For a single patient only, not recommended for multiple patients.

Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

Product material

- 1) Main body: nylon.
- 2)pulley support: polyethylene
- 3) pull ring: thermoplastic elastomer
- 4)backing strap :nylon

Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

⚠ Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which pose risk to be made worse as a result of compression of the abdomen and/or pressure.

Storage, transportation and expiry date of products

- 1)Product is valid: Three years
- 2)Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3)Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- Manufacture Date
- LOT Batch Code
- MD medical device
- CE CE Symbol
- Protect from sunlight
- Hand wash only
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

Accessories list

Instructions----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street,Longhua District,
Shenzhen, China.518109
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: info@simplemedicla.cn`

EC REP
SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

***Please refer to the Instructions before usage.**