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【网址】 www.simplemedical.cn

Simple Medical (Shenzhen) Limited Instruction For Use

Arm Sling Shoulder Brace E9

Model: 31802/31803 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

1)Product name: Arm Sling Shoulder Brace E9

2)Product model: 31802/31803

3)Product specification: F

4) Product Structure:

The product mainly consists of 45° shoulder abductor, shoulder strap, belt, bandage and Velcro

5) Operating principle

The product was designed in an open format to prevent internal rotation by fixing the patient's arm in a neutral position. The shoulder strap, belt and waist form triangle structure, which can effectively reduce the neck pressure. 45° Outward pillow arc design, more close to the waist, uniform force to make patients more comfortable.

6) Scope of application:

Suitable for shoulder soft tissue injury or postoperative use, rotator cuff tear, dislocation of shoulder joint, after shoulder or thoplasty, conservative quality of fracture, stable fracture or fracture after reduction, fracture fixation after operation.

Product features and performance description

1)Product features and storage conditions


【Product features】 shoulder joint abduction fixation, easy to fit, easy to adjust, arc design, comfortable to wear

【Storage conditions】

This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating

【Application Instructions】

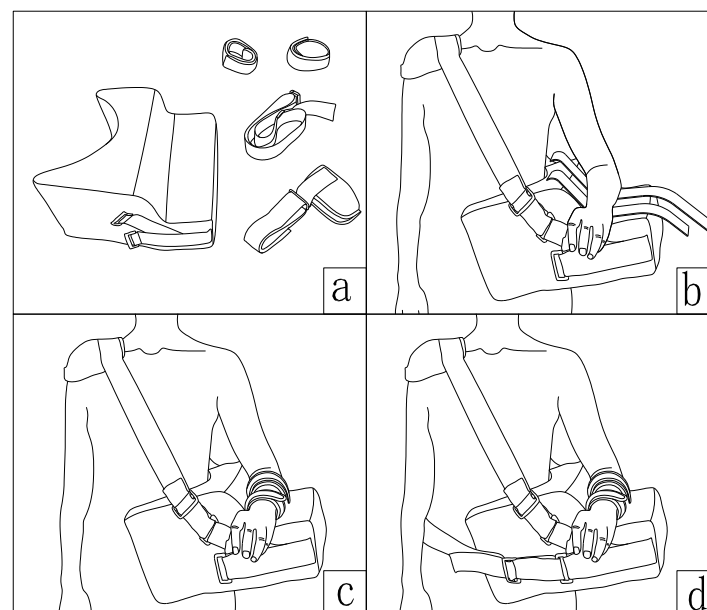APPLICATION INSTRUCTIONS

STEP 1: Open the product (Fig a)

STEP 2: First place the strap on the shoulder outer pillow and fix the side of the shoulder strap. Then put the shoulder-extension pillow to the waist, the shoulder strap around the shoulder is fixed on the other side of the shoulder-extension pillow and adjusted to the comfortable length, and the arm is placed at the shoulder-extension pillow. (Fig b).

STEP 3: Fix the strap under the arm to prevent arm movement (Fig c).

STEP 4: Fix the belt and adjust the length of the belt (Fig d)

Maintenance:

Lining can use neutral detergent and 30°C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry). If you need detergent, please use mild detergent, do not use softener and bleach. The occipital core in the external pillow of the shoulder cannot be cleaned

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors

Basic product environment

1) Ambient temperature: -15 ~ 60 °C

2) Relative humidity: 80%

Product material

1) Shoulder strap: nylon/ sponge

2) Straps: nylon

3) shoulder abduction pillow: nylon/ sponge

4) Buckle: Polyethylene

5) magic patch: nylon

Contraindication.

⚠ Do not use in case of allergy;

⚠ Those with unknown skin masses are forbidden;

⚠ Those with swollen or ulcerated skin are forbidden;

Storage, transportation and expiry date of products

1) Product is valid: Three years

2) Product storage: Please place it in a dry and clean room before use to ensure ventilation

3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation.

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

 Manufacture Date

 Batch Code

 medical device

 CE Symbol

 Hand wash only

 Protect from sunlight

 Do not use when the packing is damaged

 Warnings and Precautions

 Reference instructions

Accessories list

Instructions ----- 1 copy

Outer shoulder pillow 45°----- 1 PCS

Belt ---- 1PCS

Straps ----- 2 PCS

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited

Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street, Longhua District,
Shenzhen, China.518109

Tel : +86-755-86038563

Web: www.simplemedical.cn

E-mail: info@simplemedicla.cn`

 SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

*Please refer to the Instructions before usage