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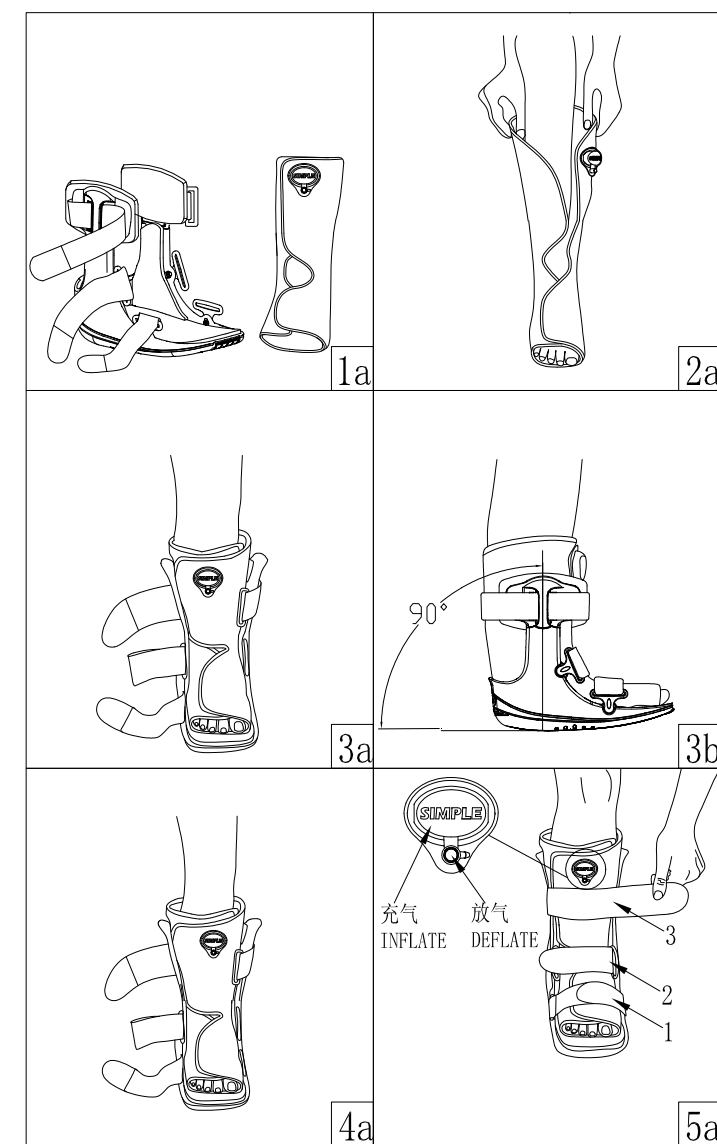
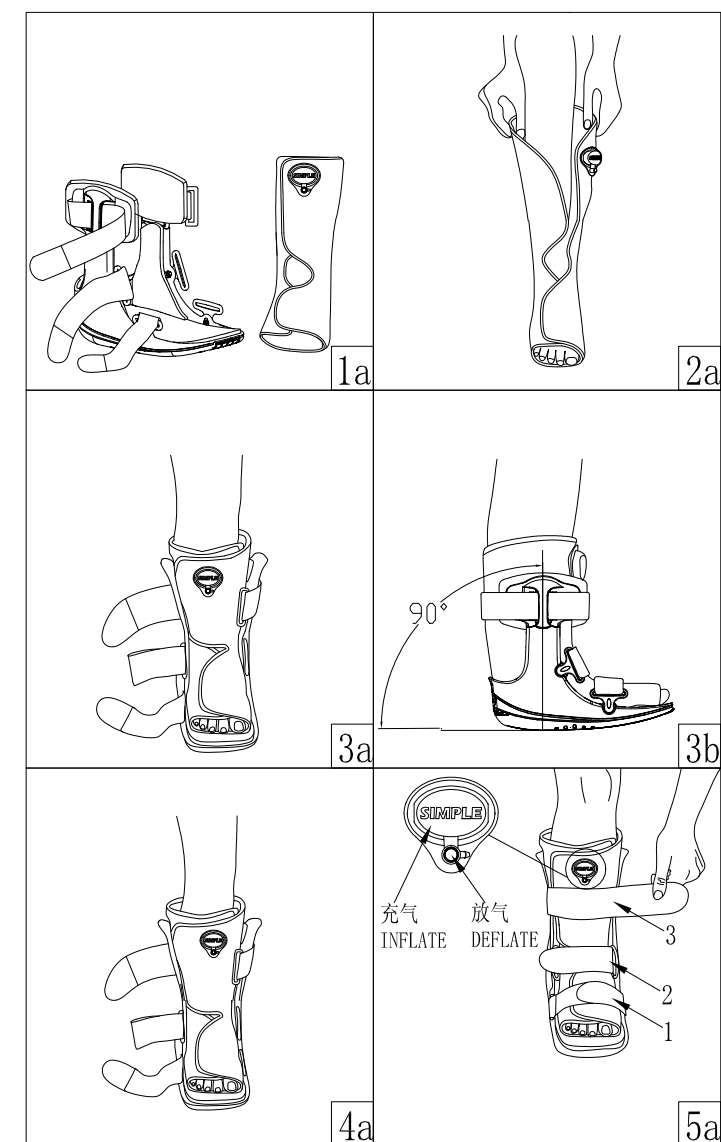
Walker B2

Model: 61002 Size: XS, S, M, L, XL

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

- 1)Product name: Walker B2
- 2)Product model: 61002
- 3)Product specification: XS, S, M, L, XL
- 4)**Product Structure:** The product is mainly composed of base, Strut, strap, lining, adhesive and bladder.
- 5)**Operating principle:**

The main body of the product is equipped with support, non-skid soft sole and bladder. The ankle joint and calf are properly protected by supporting device. Through software and shock absorption systems, patients have more comfort in their legs during exercise. The combination of the upper body of the shoe and the front cover of the calf makes the leg of the patient well protected. The breathable soft insole makes the product provide comprehensive protection to the foot before and after the use of the foot, prevent secondary injury, and provide soft contact between the calf and the fixed support to make it more comfortable to walk. The combination of soft shock absorber and anti-skid soft sole can give the patient's foot a good cushioning effect and help the patient get a good recovery.

Scope of application:

External use, provides compressed air support for foot fracture, foot ligament injury fixation, foot sole and toe injury fixation, mild and moderate foot thumb valgus, plantar pain, postoperative rehabilitation.

Product features and performance description

- 1) Product features and storage conditions
 - 【Product features】New Generation Air pump system, Low profile rocker sole, Shock-abosobing technology, High strength & light weight, Mechanical strap locking system.
 - 【Storage conditions】This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

- 2) Using the operating 【Application Instructions】
- Note1: Do not contact the liner directly with the skin.**
- *Please purchase and use the product under the guidance of your doctor or professional.**

- STEP 1:** release the nylon buckle and remove the internal gasket (Fig1a).
- STEP 2:** place the foot in the pad, heel close to the back of the pad, wrap the foot and leg. Make sure the legs and feet are close to the gasket (Fig 2a).
- STEP 3:** gently expand the side pillar and place the calf in the bracket while the pull strip is intact. The ankle should be concentrated on the pillar, with the foot and ankle at 90 °(Fig3b).
- STEP 4:** After correct alignment, adjust the straps and gaskets to know the comfortable position (Fig 4a)
- STEP 5:** Fasten the strap 1-2-3-4-5 from the bottom to the top of the toe, pay attention to the hole position of the strap and the column position on the support frame can not be separated, fasten the strap not too tight. As the swelling pattern changes in one day, it may be

desirable to adjust the degree of tightness of the strap for comfort After fixation, inflate the liner, press and release the central bubble ball until the liner provides a Comfortable cushioning and support. Do not overfill. To deflate or adjust the pressure, press the smaller circular side button (Fig5a)

STEP 6: To remove press the release button until the air is finished and then loosen the straps. Pull the heel and ankle forward. If you want to wear again, put the foot and ankle in the pad and fix the straps again.

- Note 2: 1.** Do not inflate the airbag too much. Inappropriate balloon inflation may cause significant stimulation in patients with diabetic neuropathy. If any visible skin changes occur or the patient reports discomfort, reduce the pressure on the airbag. Do not use this instrument for patients who are unable to express their discomfort.
- 2.** When walking on a slippery surface, care should be taken to avoid injury. At high altitude, the expansion of airbags will exceed the optimal level.
 - 3.** Be sure to check before using it. Do not use damaged products. If there is any tear, wear and tear in this instrument, please stop using it.

Maintenance: The software part can be washed with a neutral detergent and a 30℃ sailor and air-dried. Do not put the dryer or use other heat sources for drying. Elements such as salt water, sand, dirt, etc. may cause injury. After exposure to these conditions, rinse thoroughly with clear water, dry the towel or air-dry (do not machine the machine). For detergents, use mild detergent and do not use softeners and bleaches.

The rigid part can be scrubbed with wet cloth and dried naturally.

Warning

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

***This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.**

Basic product environment

- 1) Ambient temperature: -15 ~ 60℃
- 2) Relative humidity: 80%

Product material

- 1) Base/ strut: polypropylene+GF
- 2) Straps: nylon
- 3)Pad: nylon / polyurethane / polyester (outer net / foam / inner mesh)
- 4) Bladder/pump: thermoplasticpolyurethane
- 5)Shoecoutsole/shoe inner sole: thermoplastic elastomer / polypropylene/EVA

Contraindication

- ⚠ Use with caution for unstable fractures;
- ⚠ Use with caution in osteoporosis;
- ⚠ Use with caution for cognitive impairment;
- ⚠ Use with caution for bony ankylosis;

Storage, transportation and expiry date of products

- 1) Product is valid: five years
- 2) Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3) Transportation: the product shall be dust-proof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- Manufacture Date
- Batch Code
- It is valid for five years
- Protect from sunlight
- Hand wash only
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

Accessories list

- Instructions ----- 1 copy
- Pad ----- 3 pcs

After-sales warranty

Warranty: all products from the date of purchase as the original date, normal wear and tear of the product in the use of the process is not considered defective.

The following parts from the date of purchase to the warranty period, if there is any material or process defects, please contact your seller directly, Simple medical will complete or partial repair or replacement of equipment and accessories.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street,Longhua District,
Shenzhen, China.518131
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: info@simplemedical.cn`

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands
Tel : +31(0)10-3034500
*Please refer to the Instructions before usage.