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#### SIMPLE Ankle Brace A3

Model: 62009&62010 Size: S/M, L/XL

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

#### Product overview

- 1)Product name: SIMPLE Ankle Brace A3
- 2)Product model: 62009&62010
- 3)Product specification: S/M, L/XL
- 4)Product Structure:

The product is mainly composed of polymer shell, sponge pad, straps, rivets, magic stickers and so on.

#### 5)Operating principle

The product adopts lightweight design, easy to use and comfortable to wear. The product is a functional ankle protector, which can stabilize the upper ankle joint and help prevent foot varus and foot valgus. A solid shell in accordance with the anatomical shape of the human body provides a solid and comfortable fitting experience for patients. The built-in soft-foam pad and a specially designed flexible edge help to prevent pain points and provide a comfortable feel to the patient. Adjustable straps are easy for patients to use. The structure of the product is slender and can be worn in all kinds of shoes

#### 6)Scope of application

- Ligament and capsule injuries at the upper ankle joint (e.g. ligament ruptures, strains, distortions)
- Slight soft tissue injuries
- Chronic ankle instability
- Rehabilitation and prevention of ankle injuries
- Acute, chronic, and inflammatory edema

#### Product features and performance description

##### 1) Product features and storage conditions


**【 Product features 】** The product is convenient to use and comfortable, has good air permeability, high strength, good fixation and can be cleaned.

**【 Storage conditions 】** This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

##### 2)Using the operating

**【 Application Instructions 】**

#### APPLICATION INSTRUCTIONS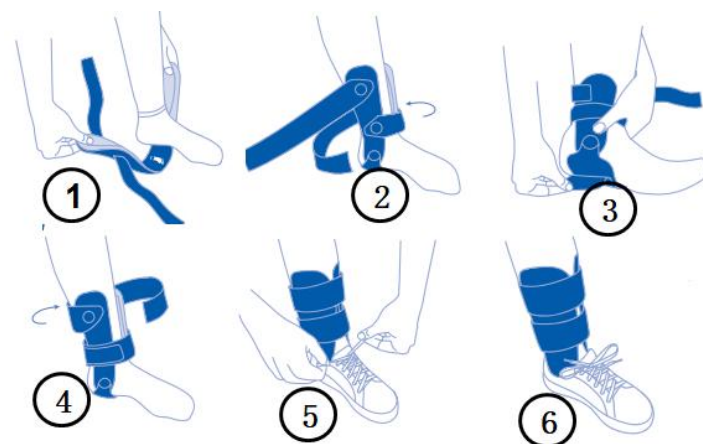

**Note: for the best wearing comfort, it is recommended to wear socks and then wear protective gear.**

**STEP 1:** place the foot on the heel strap between the two shells (Fig 1)

**STEP 2:** fix the lower strap (Fig2).

**STEP 3:** open the heel strap and adjust to a comfortable width(Fig 3)

**STEP 4:** fix the upper strap (Fig4).

**STEP 5:** put on your shoes and get the best support at normal gait (Fig 5 & 6).

#### Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

All parts must be air-dried and completely dry before use

#### Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

\*This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

\*For Single Patient Use Only

#### Basic product environment

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

#### Product material

- 1) Housing: ABS+PC/TPE
- 2) Pad: foam
- 3) Rivet: stainless steel
- 4) Strap: nylon

#### Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

#### Storage, transportation and expiry date of products

- 1)Product is valid: five years
- 2 )Product storage : Please put in a dry and clean room for indoor ventilation before use. Do not mix with corrosive substances such as acid and alkali and keep away from the fire source.
- 3)Transportation: the product shall be dustproof, moisture-proof and pollution proof during storage and transportation

#### Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

#### Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- CE Symbol
- Hand wash only
- Protect from sunlight
- Do not use when the packing is damaged
- Warnings and Precautions
- It is valid for five years
- Reference instructions

#### Accessories list

Instructions ----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street, Longhua District, Shenzhen, China.518131  
Tel : +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: service@simplegroup.net

SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

\*Please refer to the Instructions before usage

62009&10 V0.2