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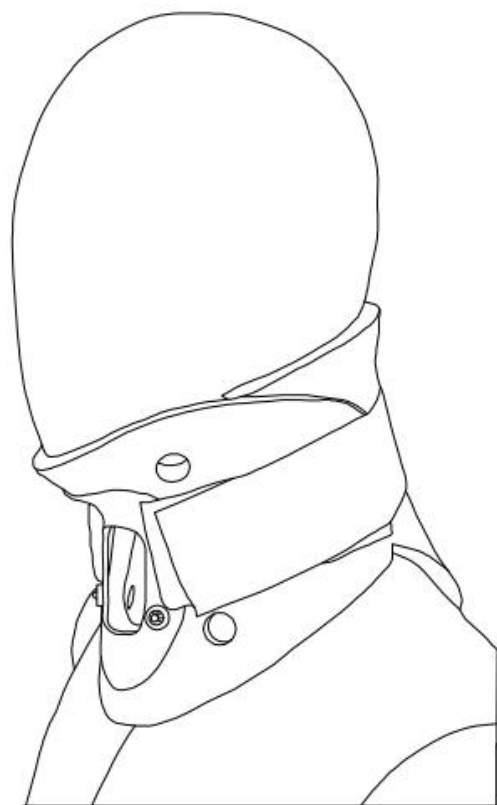
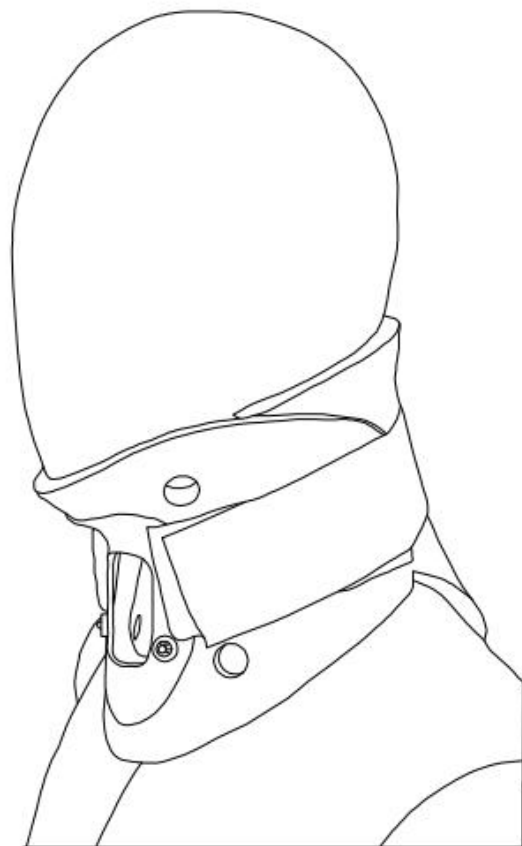
SIMPLE Cervical Collar C1

Model: 51002 Size: S, M, L

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference.

Product overview

- 1)Product name: SIMPLE Cervical Collar C1
- 2)Product model: 51002
- 3)Product specification: S, M, L
- 4)Product Structure:

The product is usually made of polymer materials, fabrics, metals and other materials.

5)Operating principle:

The product adopts a streamlined and lightweight design, conforming to biomechanical and ergonomic design. Special curved cutting, so that the neck has a full range of stability and comfort. The multi-point support force is uniform, as if tailor-made comfort.

6)Intended Use:

Used for neck repositioning fixation.

7)Adaptation disease:

It is suitable for the prevention and auxiliary treatment of cervical spondylosis, cervical muscle strain and other cervical spine syndromes and neck fixation during the rehabilitation period after cervical spondylosis.

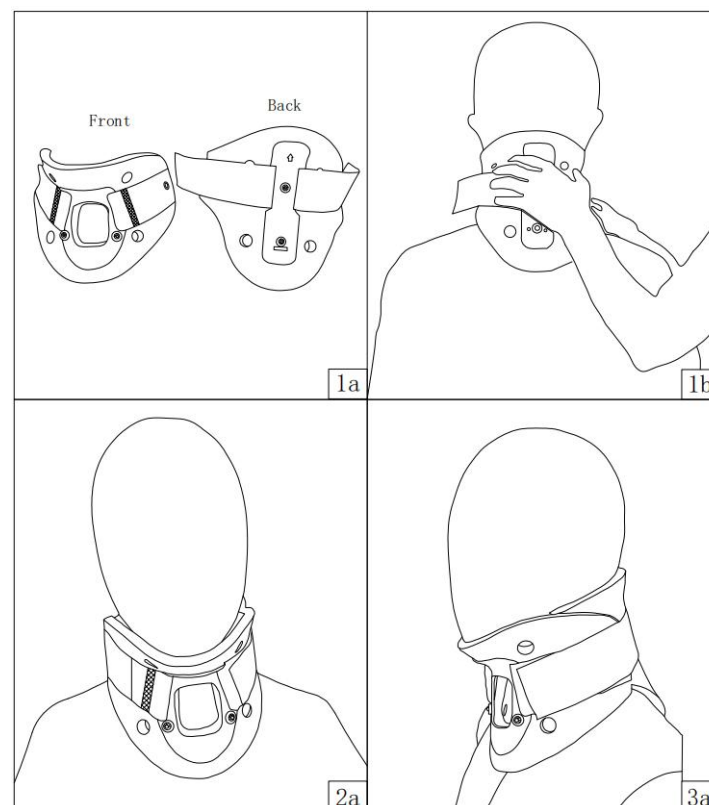
Product features and performance description

1) Product features and storage conditions

【 Product features 】 Safe, environmental protection, light weight, with air holes, comfortable and convenient to wear.

【 Storage conditions 】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating

【Application Instructions】

APPLICATION INSTRUCTIONS

STEP 1:

Insert the back piece neck brace carefully into the back neck.(Fig 1a、1b)

STEP 2:

The jaw pad of the front neck support is fitted to the jaw, the front piece is pressed against the back piece, and the Velcro is fixed.(Fig. 2a)

STEP 3:

Check whether the product is worn correctly and comfortably. (Fig 3a)

Note: Use it under the guidance of a doctor or a professional.

Notes

- 1.Always check whether each part is firm, whether the hook buckle, etc. are safe and reliable,If a problem is found, it should be repaired or replaced in time.
2. The training time should be according to the instructions of the doctor or rehabilitation trainer.
3. If pain or paresthesia occurs during use, please stop using immediately and contact your doctor.

Maintenance

The product is hand-washed with clear water and is naturally air-dried. Do not put the dryer or use other heat sources for drying. Elements such as salt water, sand, dirt, etc. may cause injury. After exposure to these conditions, rinse thoroughly with clear water, dry the towel or air-dry (do not machine the machine). For detergents, use mild detergent and do not use softeners and bleaches.

Basic product environment

- 1) Ambient temperature: -15 ~ 60℃
- 2) Relative humidity: 80%

Product material

- 1) Body material: polypropylene
- 2) Liners: sponge
- 3) Strap: nylon / elastic tape / magic sticker

Contraindication

- ⚠ Those with bleeding tendency should be used with caution.
- ⚠ Patients with malignant tumors and hyperthermia should be used with caution.
- ⚠ Patients with severe hypertension and cardiovascular disease should be used with caution.
- ⚠ People with severe skin allergies should use with caution.

Storage, transportation and expiry date of products

- 1)Product is valid: Three years
- 2)Product storage: Please place it in a dry and clean room before use to ensure ventilation
- 3)Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

- ⚠ Attention, warning
- 🧼 Hand wash only
- ☀️ Protect from sunlight
- 🚫 Do not use when the packing is damaged
- 🕒 It is valid for three years
- 📖 Reference instructions

Accessories list

Instructions ----- 1 copy

After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

Simple Medical (Shenzhen) Limited
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street,Longhua District,
Shenzhen, China.518131
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: service@simplegroup.net

SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

***Please refer to the Instructions before usage**