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**【网址】**www.simplemedical.cn

**Simple Medical (Shenzhen) Limited**  
**Instruction for Use**

**SIMPLE Wrist brace W3**

Model:42001

Size: XS,S,L

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

**Product overview**

- 1)Product name: SIMPLE Wrist brace W3
- 2)product model: 42001
- 3)Product specification: XS,S,L
- 4)Product Structure:

The wrist fixed support mainly by breathable soft material, aluminum alloy, plastic parts, fixed belt

5)Operating principle:

The lightweight design of the product enhances patient comfort through supports and breathable and soft fabrics. The support piece fits the wrist contour, so that the wrist and guard have proper support and protection. Help relieve wrist muscle and joint pain, provide support and protection for injured wrists, avoid re-injury, and effectively ensure that patients get a good recovery.

6) Adapt to symptoms:

Wrist sprain. Non surgical wrist fracture. Post-operative or post-traumatic wrist immobilisation. Wrist tendinopathies. Conservative treatment of carpal tunnel syndrome. Rheumatoid arthritis. Wrist osteoarthritis.

**Product features and performance description**

1) Product features and storage conditions


**【 Product features 】** Ergonomic design, breathable and lightweight, easy to wear and easy to take off, stable support, strengthen the wrist fixed, left and right hand universal.

**【Storage conditions】** This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2)Using the operating

**【Application Instructions】**

APPLICATION INSTRUCTIONS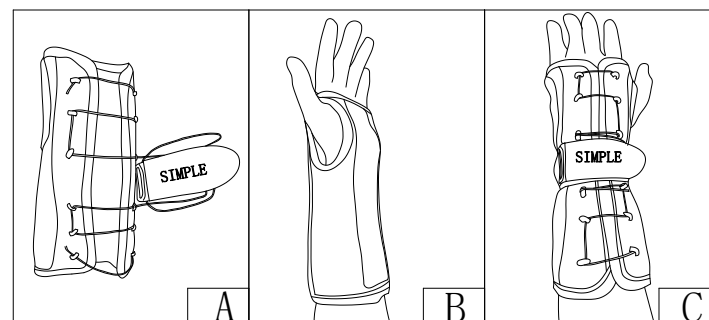

**STEP 1.**

Remove all the straps of the wrist joint retainer and reach into the wrist joint retainer (Figure A&B).

**STEP 2.**

Tighten the straps, adjust the straps according to comfort and support, and, if necessary, adjust the protective lining when tightened.(Figure C).

**STEP3.**

If necessary, adjust the protective lining during tightening.

**Note: If worn too tightly, it may cause skin irritation or pressure. Please consult before wearing and wear under the guidance of a doctor.**

**Maintenance:**

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

**Warning:**

If you experience any uncomfortable reaction such as pain, swelling, allergies, sensory disturbances, poor blood flow, etc. while using this product, discontinue use and consult a doctor.

**Product basic parameter**

- 1) Ambient temperature: -15 ~ 60 °C
- 2) Relative humidity: 80%

**Product materials**

- 1) Body material: Polyester, nylon
- 2) Support sheet: Aluminum + Plastic
- 3) Strap: Nylon/Polyester

**Contraindication**

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;
- ⚠ Unstable fractures of the hand;

**Storage, transportation and expiry date of products**

- 1)Product is valid: Five years.
- 2 )Product storage: Please place it in a dry and clean room before use to ensure ventilation.
- 3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation.

**Disposal of wastes**

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

**Symbol and picture description**

- ⚠ Attention, warning
- ☒ Hand wash only
- ☀ Protect from sunlight

- ☒ Do not use when the packing is damaged
- ⌚ It is valid for five years
- 📖 Reference instructions

**Accessories list**

Instructions ----- 1 copy

**Selection**

<b>Item</b>	42001	
<b>Size</b>	XS, S, L	
	<b>Size</b>	<b>Measurement method</b>
<b>XS</b>	110-130mm	
<b>S</b>	130-153mm	
<b>L</b>	178-203mm	

**After-sales warranty**

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street, Longhua District, Shenzhen, China.518131  
Tel : +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: service@simplegroup.net

**EC REP** SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands  
Tel : +31(0)10-3034500

\*Please refer to the Instructions before usage

**CE** 42001 V0.2