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## Simple Medical (Shenzhen) Limited Instruction For Use

SIMPLE Hip Brace N3

Model: 21003 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference.

### Product overview

- 1) Product name: SIMPLE Hip Brace N3
- 2) Product model: 21003
- 3) Product specification: F
- 4) **Product Structure**

It is composed of aluminum alloy, stainless steel, sponge and plastic. It is worn on the body surface of the body to achieve the purpose of maintaining the stability of the body by restricting the movement of the body. It has no orthopedic function.

### 5) Operating principle

The product has an open design, and the belt and hip support combine to effectively reduce pressure on the hip. It fits the waist better and applies even force to make the patient more comfortable.

### 6) Scope of application:

Post-Op use following minimally invasive hip procedures; Labral repair, with or without gluteus medius repair; ROM control following primary or hip revision surgery; Post-op use following proximal hamstring repair.

### Product features and performance description

#### 1) Product features and storage conditions

**【Product features】** The product is easy to use and comfortable, the force is uniform, and the tightness is adjustable.

**【Storage conditions】** This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature.

#### 2) Using the operating

### APPLICATION INSTRUCTIONS

**Step 1:** Before wearing, unroll the belt and leg fastener, pass the belt through the brace slot, and adjust the belt to fit the waist. (Figure 1a&2a&3a)

**Step 2:** Use pull loops to fine-tune the belt size to fit the waist. (Figure 1b)

**Step 3:** Step 3: Position the pelvic shell and articulation device so that it is centered on the side.

**Step 4:** Adjust the length of the rod. Press and hold the adjusting button on the support and push the support to the required length. Release it to lock the plastic support.

**Step 5:** Adjust hip range of motion.. Press and hold down the Angle adjustment button of flexion or extension, move it to the marked Angle scale as needed, and press the knob of the re-angle adjustment button to the inside of the leg to 90° to lock it. (Figure 2c)

**Step 6:** Tighten leg straps. Hold the retainer in place by hand to prevent movement. And fasten the buckle in turn, buckle the buckle in place. After securing the strap position, adjust the strap length and cushion comfort before tightening the strap. (Figure 3d)

### The strap length is adjusted

Determine the required length, open the Velcro at the end of the strap, separate the Velcro and cut the strap to the required length, and then attach the strap back to the end of the strap.

### Outreach/adduction instructions

If needed, the clinician can change the abduction/adduction Angle using the wrench tool provided.

Time to wear the retractor after surgery: The standard for wearing the retractor should be determined by the doctor to determine the Angle of hinge support and flexion and the time to wear it each day until your primary care doctor gives you permission to remove the retractor.

### Maintenance:

Lining can use neutral detergent and 30℃ water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach. The occipital core in the external pillow of the shoulder cannot be cleaned

### Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

\*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors.

### Basic product environment

- 1) Ambient temperature: -15 ~ 60 ℃
- 2) Relative humidity: 80%

### Product material

- 1) Belt: Nylon
- 2) Bracket buckle: polyethylene
- 3) Velcro: Nylon
- 4) Connecting rod: stainless steel

### Contraindication

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;

⚠ Do not use for indications greater than 45° pronation or abduction.


### Storage, transportation and expiry date of products

- 1) Product is valid: Five years
- 2) Product storage: Please place it in a dry and clean room before use to ensure ventilation.
- 3) Transportation: The product shall be dustproof, moisture-proof and pollution-proof during storage and transportation.

### Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

### Symbol and picture description

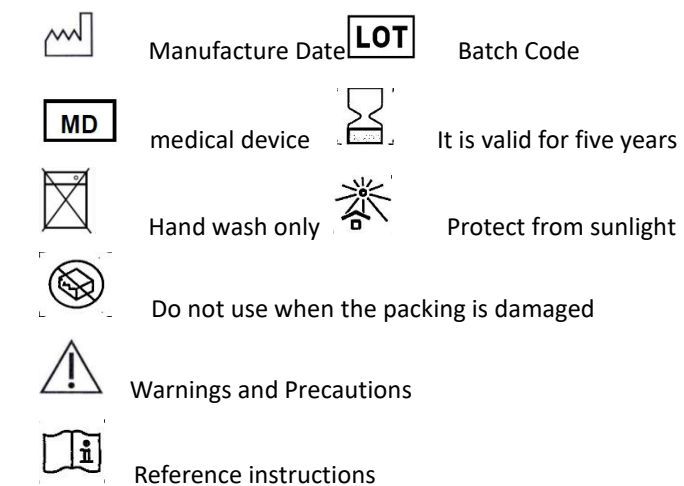### Accessories list

Instructions ----- 1 copy

### After-sales warranty

If there is any defect in the material or process of the product, please contact your vendor directly, and the Simple medical will repair or replace the product.

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road,  
Minzhi Street, Longhua District,  
Shenzhen, China.518131  
Tel: +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: service@simplegroup.net

Fascinatia Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel: +31(0)10-3034500

\*Please refer to the Instructions before usage.

21003 V0.2