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SIMPLE Shoulder Brace E3

Model: 31002 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

Product overview

1)Product name: SIMPLE Shoulder Brace E3

2)Product model: 31002

3)Product specification: F

4) Product Structure

The products are made of polyester fiber and nonwoven fabric. Not connected to medical devices used in the body. Non-sterile delivery.

5)Operating principle

The product adopts open design and soft and breathable material, making the product easy to use and comfortable to wear. The reinforced chest strap makes the arm not easy to move with double protection.

6)Scope of application:

Nonsurgical treatment after anterior shoulder dislocation and /or chronic subluxation, After rotator cuff repair, Acromioclavicular joint dislocation (nonsurgical, postoperative), Scapula fractures (nonsurgical, postoperative), Fractures of the humerus head (nonsurgical, postoperative).

Product features and performance description

1) Product features and storage conditions

【Product features】 The product is comfortable and breathable, easy to wear, easy to adjust, left and right arms universal.

【Storage conditions】 This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature.

2) Using the operating **【Application Instructions】**

 Step 1: Open the sling before wearing and place the injured arm into the open sling. (Fig 1a)

Step 2: Apply a magic fit to the sling. (Fig 1a~2b)

 Step 3: Put your uninjured arm through the collar consisting of Strap 1, strap 2, and strap 3, pull strap 1 to your chest, close the buckle of strap 1, and adjust the shoulder strap to the appropriate position. (Fig 3c)

 **Note: If possible, take off and put on at the table. If the AIDS are worn too tightly, they may create local pressure or compress blood vessels or nerves. Therefore, please wear it on the advice of your attending physician.**

Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach.

Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors.

Basic product environment

1) Ambient temperature: -15 ~ 60 °C

2) Relative humidity: 80%

Product material

1)chest strap / shoulder strap: Polyester

2) shoulder pad: non woven fabric

3) big net pocket: Polyester

Contraindication

 Do not use in case of allergy;

 Those with unknown skin masses are forbidden;

 Those with swollen or ulcerated skin are forbidden;

 Do not use indications that require abduction and external rotation.

Storage, transportation and expiry date of products

1)Product is valid: Five years

2)Product storage: Please place it in a dry and clean room before use to ensure ventilation

3)Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation

Disposal of wastes

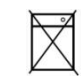
Disposal of wrapping paper (bags) : please dispose of the packaging of this product according to the local laws.

Symbol and picture description

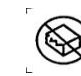
 Manufacture Date

 Batch Code

 medical device

 Hand wash only

 Protect from sunlight

 Do not use when the packing is damaged

 Warnings and Precautions

 Expiry Date

 Reference instructions

Accessories list

Instructions ----- 1 copy

Warranty

Simple Medical (Shenzhen) Limited warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Simple Medical (Shenzhen) Limited in the event of breach of warranty shall be to repair or replace the defective product or part(s).

 **Simple Medical (Shenzhen) Limited**
Address: 519 Wisdom Valley, No.1010 Bulong Road,
Minzhi Street,Longhua District,
Shenzhen, China.518131
Tel : +86-755-86038563
Web: www.simplemedical.cn
E-mail: service@simplegroup.net

 SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands
Tel : +31(0)10-3034500

***Please refer to the Instructions before usage**

 **31002 V0.3**