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【网址】www.simplemedical.cn

## Simple Medical (Shenzhen) Limited Instruction For Use

SIMPLE Shoulder Brace E5

Model: 31001 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

### Product overview

1)Product name: SIMPLE Shoulder Brace E5

2)Product model: 31001

3)Product specification: F

### 4) Product Structure:

It is made of polyethylene, nylon, sponge and other materials. Non-sterile delivery, not connected to medical devices used in the body.

### 5) Operating principle

The product is open-ended designed to prevent internal rotation by fixing the patient's arm in a neutral position. The shoulder belt, the waistband and the waist form a triangular structure, and the neck pressure is effectively reduced. The detachable holding force rehabilitation ball is convenient to use and can exercise the gripping force of the hand finger at any time. The strap and the belt are in a snap-fit connection to make the patient wear more convenient.

### 6) Scope of application:

It is suitable for trigonometry, upper tendon rupture, shoulder and arm joint fracture and dislocation, shoulder and arm joint operation support, brachial plexus injury.

### Product features and performance description

1)Product features and storage conditions

#### 【Product features】


Fixation with shoulder abduction, Easy fit and convenient adjust

#### 【Storage conditions】

This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature

2) Using the operating

#### 【Application Instructions】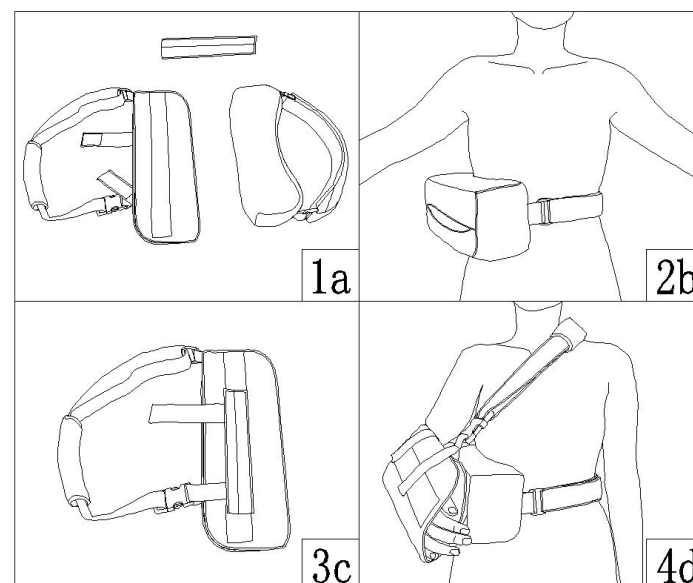

### APPLICATION INSTRUCTIONS

**STEP 1:** Open the product (Fig. 1a)

**STEP 2:** Loosen the buckle of the belt, place the shoulder pillow at the waist, and connect the buckle. Adjust the length of the belt so that the belt fits with the waist (Fig. 2b).

**STEP 3:** Stick the double-sided magic sticker on the hair surface of the main pocket near one side of the body (Fig. 3 c).

**STEP 4:** Loosen the two straps and shoulder straps on the outside of the sling pocket, place the hand in the sling pocket, fix the fastening belt on the hairy surface of the sling pocket, place the sling on the shoulder spread pillow, connect the shoulder strap buckle through the shoulder, and adjust the shoulder belt to the appropriate length so that the shoulder pad is located at the shoulder (Fig4 d).

### Maintenance:

Lining can use neutral detergent and 30 °C water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry).If you need detergent, please use mild detergent, do not use softener and bleach. The ccipital core in the external pillow of the shoulder cannot be cleaned

### Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

\*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors

### Basic product environment

1) Ambient temperature: -15 ~ 60 °C

2) Relative humidity: 80%

### Product material

1) Shoulder strap: nylon/ sponge

2) Main body suspension: nylon/ sponge

3) shoulder abduction pillow: nylon/ sponge

4) Buckle: Polyethylene

5) grip ball: EVA

6) Double-sided magic patch: nylon

### Contraindication.

⚠ Do not use in case of allergy;

⚠ Those with unknown skin masses are forbidden;

⚠ Those with swollen or ulcerated skin are forbidden;

### Storage, transportation and expiry date of products

1)Product is valid: Three years


2 )Product storage: Please place it in a dry and clean room before use to ensure ventilation

3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation.

### Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

### Symbol and picture description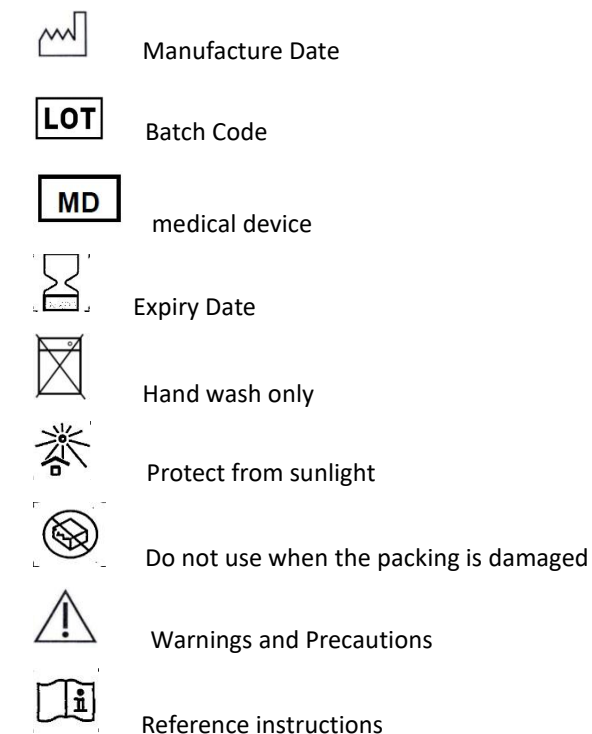

### Accessories list

Instructions ----- 1 copy

Outer shoulder pillow 30°----- 1 PCS

Grip ball----- 1 PCS

Double sided magic sticker----- 1 PCS

### Warranty

Simple Medical (Shenzhen) Limited warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Simple Medical (Shenzhen) Limited in the event of breach of warranty shall be to repair or replace the defective product or part(s).

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road,  
Minzhi Street, Longhua District,  
Shenzhen, China.518131  
Tel : +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: service@simplegroup.net

**EC REP**  
SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel : +31(0)10-3034500

\*Please refer to the Instructions before usage

**CE** 31001 V0.2