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【网址】www.simplemedical.cn

## Simple Medical (Shenzhen) Limited Instruction For Use

SIMPLE Shoulder Brace E13

Model: 31013 Size: F

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference

### Product overview

- 1) Product name: SIMPLE Shoulder Brace E13
- 2) Product model: 31013
- 3) Product specification: F
- 4) **Product Structure:**

Products are mainly composed of 15° external rotation shoulder strap, shoulder strap, belt, bandage, Velcro and so on.

### 5) Operating principle

The product has an open design that prevents internal rotation by holding the patient's arm in a neutral position. The shoulder strap, belt and waist form a triangular structure to effectively reduce neck pressure. 15° external rotary pillow arc design, more fit waist, uniform force to make patients more comfortable.

### 6) Scope of application:

After rotator cuff repair, Humeral head fractures (nonsurgical, Postoperative), After insertion of shoulder prosthesis, After stabilization of acromioclavicular joint dislocations, After biceps tenodesis, After biceps tenodesis, Impingement syndrome (nonsurgical, postoperative).

### Product features and performance description

#### 1) Product features and storage conditions

**【Product features】** Nylon buckle connection, easy to adjust the abduction Angle, ergonomically designed arc sponge, comfortable to wear, widened shoulder belt belt design, with breathable shoulder pad wear no tight.

**【Storage conditions】** This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature.

#### 2) Using the operating

#### 【Application Instructions】

1a

2a

3a

#### APPLICATION INSTRUCTIONS

**STEP 1:** Open the buckle and sling of the cushion before wearing, and place the injured arm into the open sling. (Fig 1a)

**STEP 2:** Use Velcro to close the suspenders, pull the belt #4 of the shoulder pillow from the back to the front, adjust the appropriate elastic position, and close the buckle. (Fig1a~ 2a)

**STEP 3:** Put your uninjured arm through the collar consisting of Strap 1, strap 2, and strap 3, pull strap 1 to your chest, close the buckle of strap 1, and adjust the shoulder strap to the appropriate position. (Fig 3a)

**Note:** If possible, take off and put on at the table. If the AIDS are worn too tightly, they may create local pressure or compress blood vessels or nerves. Therefore, please wear it on the advice of your attending physician.

### Maintenance:

Lining can use neutral detergent and 30℃ water washed by hand, and natural dry. Do not use clothes dryer or other heat source for drying. Salt water, sand, dirt and other elements can cause damage. After exposure to these conditions, rinse thoroughly with water, dry with a towel or air dry (do not machine dry). If you need detergent, please use mild detergent, do not use softener and bleach. The occipital core in the external pillow of the shoulder cannot be cleaned

### Warning:

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.

\*This product is only a fixture and is not used to prevent injury or prevent recurrence of injury, much less to replace the correct diagnosis of drugs and doctors.

### Basic product environment

- 1) Ambient temperature: -15 ~ 60 ℃
- 2) Relative humidity: 80%

### Product material

- 1) Shoulder strap: nylon/ sponge
- 2) shoulder pillow: nylon/ sponge
- 3) Buckle: Polyethylene
- 4) magic patch: nylon
- 5) Sling: sponge

### Contraindication.

- ⚠ Do not use in case of allergy;
- ⚠ Those with unknown skin masses are forbidden;
- ⚠ Those with swollen or ulcerated skin are forbidden;
- ⚠ Do not use for indications greater than 15° pronation or abduction.

### Storage, transportation and expiry date of products

- 1) Product is valid: Five years
- 2) Product storage: Please place it in a dry and clean room before use to ensure ventilation.
- 3) Transportation: the product shall be dustproof, moisture-proof and pollution-proof during storage and transportation.

### Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

### Symbol and picture description

- Manufacture Date
- Batch Code
- medical device
- Expiry Date
- Hand wash only
- Protect from sunlight
- Do not use when the packing is damaged
- Warnings and Precautions
- Reference instructions

### Accessories list

Instructions ----- 1 copy

### Warranty

Simple Medical (Shenzhen) Limited warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Simple Medical (Shenzhen) Limited in the event of breach of warranty shall be to repair or replace the defective product or part(s).

**Simple Medical (Shenzhen) Limited**  
Address: 519 Wisdom Valley, No.1010 Bulong Road, Minzhi Street, Longhua District, Shenzhen, China.518131  
Tel: +86-755-86038563  
Web: www.simplemedical.cn  
E-mail: service@simplegroup.net

SUNGO Europe B.V.

Fascinatio Boulevard 522 ,Unit1.7,2909VA Capelle aan den IJssel ,The Netherlands

Tel: +31(0)10-3034500

\*Please refer to the Instructions before usage

31013 V0.1